

# AM-1 Acupuncture Training Pad



SAKAMOTO MODEL  
CORPORATION

User's Manual (151218)

## — I N D E X —

- Introduction/Table of Contents 1
- Purpose/Content/Instructions for Use 2
- Cautions on Use/Storage Instructions 3

<b>!</b> Note	Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.
------------------	--

株式会社坂本モデル



## Introduction

---

Thank you for purchasing our product, “Acupuncture Training Pad.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

---


Introduction／Table of Contents.....	1
Purpose／Content／Instructions for Use.....	2
Cautions on Use／Storage Instructions.....	3

## **Purpose**

Acupuncture training to practice using the  
“dominant hand to hold the needle,”  
“non-dominant hand to assist insertion,”  
“perpendicular insertion” and “oblique insertion”

## **Content**

- 1) Acupuncture training pad – Qty. 5
- 2) Training belt – Qty. 5

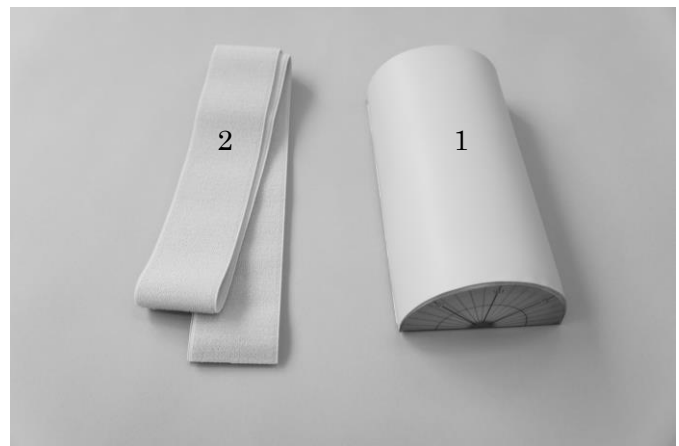

## **Instructions for Use**

- 1) When training alone, set up the training pad on a table. When training in a pair, wrap the belt around the location or area of the body where you wish to practice acupuncture.
- 2) Set the training pad onto the belt.
- 3) Begin practicing acupuncture.

## **Cautions on Use**

- 1) Use new needle tips when performing acupuncture training with this product. Otherwise, it can cause the skin of the pad or the sensor to deteriorate.
- 2) Be careful when handling the training product because the product material gets dirty easily. In addition, be particularly careful with ballpoint pens and magic markers because their marks cannot be removed.
- 3) Take extra care when handling the product because it is fragile and will break if dropped.
- 4) If the product gets dirty, use a neutral detergent on a soft cloth to remove the dirt or stain. (Do not use solvents such as benzine, thinner or polishing powder.)
- 5) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 6) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 7) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

## **Storage Instructions**

- 1) If the product gets dirty, use a neutral detergent on a soft cloth to remove the dirt or stain.
- 2) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.

---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

Head office

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:<http://sakamoto-model.co.jp>

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com