

# AM-4

## Acupuncture Model of Sciatic Nerve



SAKAMOTO MODEL  
CORPORATION

### User's Manual (151218)

#### — I N D E X —

●Introduction/Table of Contents	1
●Purpose/Content	2
●Cautions on Use/Storage Instructions	3
●Instructions for Use	4

#### ! Note

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.

株式会社坂本モデル



## Introduction

Thank you for purchasing our product, “Acupuncture Model of Sciatic Nerve.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

Introduction / Table of Contents.....	1
Purpose / Content.....	2
Cautions on Use / Storage Instructions.....	3
Instructions for Use.....	4

## **Purpose**

The following acupuncture training can be performed on the sciatic nerve.

(Technique 1) Line between posterior superior iliac spine and greater trochanter


Find a middle point that connects the posterior superior iliac spine and the greater trochanter. Then, set the insertion point 3 cm directly under the middle point.

(Technique 2) Line between ischial tuberosity and greater trochanter

Find the line that connects the greater trochanter and ischial tuberosity. Then, set the insertion point about 1/3 the length up from the ischial tuberosity side.

## **Content**

- 1) Training model
- 2) Batteries (AA alkaline batteries, Qty. 4) - 1 set
- 3) Stands for placing the model in a lateral position - 1 set

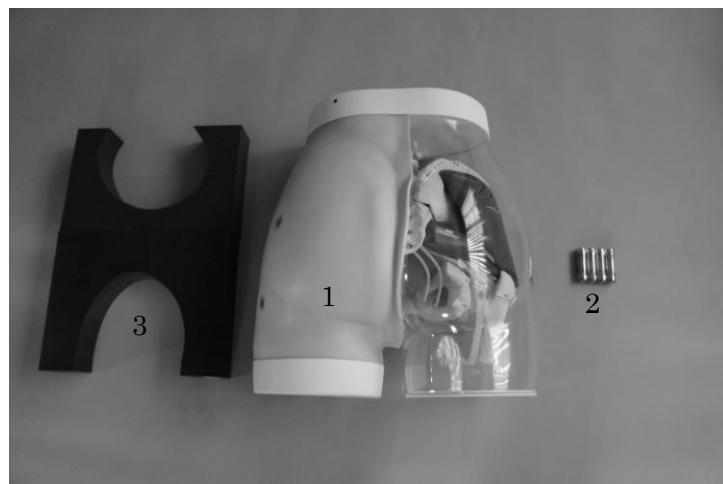

## **Cautions on Use**


- 1) Use new needle tips when performing acupuncture training with this product.  
Otherwise, it can cause the skin of the model or the sensor to deteriorate.
- 2) Take extra care when handling the product because it is fragile and will break if dropped.
- 3) If the product gets dirty, use a neutral detergent on a soft cloth to remove the dirt or stain.  
(Do not use solvents such as benzene, thinner or polishing powder.)
- 4) Use the four AA size alkaline batteries.  
Make sure that the positive and negative sides on the batteries are inserted correctly.  
Replace the batteries with new ones when the buzzer sound becomes weak or when the indicator light fades.
- 5) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 6) This product is a simulator capable of supporting advanced medical training.  
However, Sakamoto Model does not guarantee the same results during actual treatment.
- 7) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

## **Storage Instructions**

- 1) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.
- 2) Always remove the batteries when storing the product.  
Be careful when storing because if the batteries are not removed during storage, the batteries could corrode and prevent the power from turning on.

## **Instructions for Use**

- 1) Insert the four AA alkaline batteries into the battery compartment and ensure that they are inserted correctly.
- 2) Turn the power switch on.
- 3) Set up the stands provided to place the model in a lateral position.  
(Refer to the picture on the right)

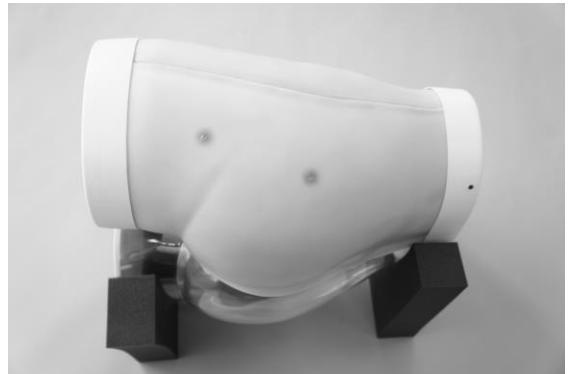

- 4) Begin practicing acupuncture.
- 5) If the point of insertion is correct, the green indicator light turns on and a buzzer sounds.

---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

**Head office**

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

**Overseas Sales Department**

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com