

M152-3

SAKAMOTO Gluteus Injection Trainer



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

| | |
|--|---|
| ● Introduction/ Table of Contents | 1 |
| ● Purpose/ Content | 2 |
| ● Cautions on Use | 3 |
| ● Instructions for Use/ Storage Instructions | 4 |

| | |
|-------------------------|--|
| ! Note | Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor. |
|-------------------------|--|

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “SAKAMOTO Gluteus Injection Trainer.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

| | |
|--|---|
| Introduction／Table of Contents..... | 1 |
| Purpose／Content..... | 2 |
| Cautions on Use..... | 3 |
| Instructions for Use／Storage Instructions..... | 4 |


Purpose

This product is used to perform the following injection training:

- 1) Evaluates correct intramuscular gluteus injection
(Hochstetter's point and Clarke's points)

Content

- 1) Trainer model (including drain bag)
- 2) Cover to check the injection site – Qty. 1
- 3) Draining syringe – Qty. 1
- 4) Batteries (AA size alkaline batteries) – Qty. 4 (1 set)

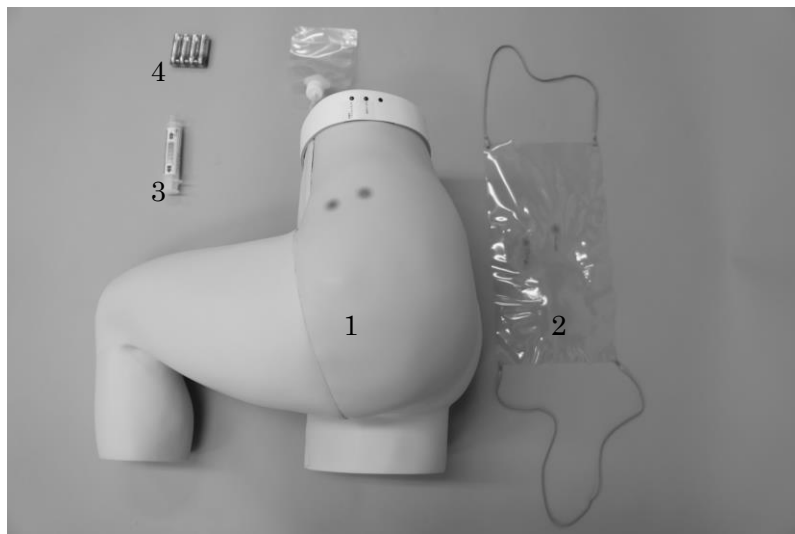

Cautions on Use

- 1) Use new needle tips when performing injection training with this product.
Otherwise, it may cause the skin of the trainer model to deteriorate.
- 2) Only use water (tap water) when training with this product.
Do not use any other solutions besides water.
- 3) Pouring water is allowed only when the green lamp is lit.
In the other cases, never pour water.
- 4) Handle this product with care because it is fragile and may break if dropped.
- 5) If the product gets dirty, use kitchen detergent or soap on a soft cloth to remove the dirt or stain.
*Do not use solvents such as benzine, thinner or polishing powder.
- 6) Use the four AA size alkaline batteries.
Make sure that the positive and negative sides on the batteries are inserted correctly.
Replace the batteries with new ones when the buzzer sound becomes weak or when the indicator light fades.
- 7) This product does not support practicing the injection technique locating the upper outer quadrant of the buttock.
- 8) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 9) This product is a simulator capable of supporting advanced medical training.
However, Sakamoto Model does not guarantee the same results during actual treatment.
- 10) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

Instructions for Use

- 1) To begin, insert the batteries into the training model.
- 2) Turn the model's power switch on.
- 3) Always be sure to use water (tap water) when injecting fluid.
Water can be injected only when the green indicator light is on.
- 4) Use this product to carry out training.
- 5) The built-in indicator light and buzzer evaluate whether the gluteus injection was performed correctly or not.
 - The green indicator light turns on when the injection is performed at the correct site, including the proper depth.
 - The red indicator light flashes and the buzzer sounds if the injection is performed at the wrong site.
 - Both the red and green indicator lights flash and the buzzer sounds if the injection is performed at the correct site but too deeply.
- 6) After training with this product, remove the drain bag from the base and dispose of the water inside the bag before storing.
- 7) When necessary, use the draining syringe to completely discharge the water, if water drops remain inside the tube.

Storage Instructions

- 1) Most of the water will have been discharged after disposing of the water inside the drain bag. However, if you are concerned about any moisture that remains inside the drain tube, the draining syringe that is provided in the accessories can be used to remove it.
- 2) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.
- 3) Always remove the batteries when storing the model.
Be careful when storing because if the batteries are not removed during storage, the batteries could corrode and disrupt the power supply.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

**34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN**

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

**1-9-29,Senbahigashi,Minoh City,Osaka
562-0035,JAPAN**

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com