

M160

Transparent Female Catheter Model

M160-1

Transparent Male Catheter Model



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

- Introduction/ Table of Contents 1
- Purpose/ Content 2
- Precautions Concerning handling 3
- Usage 4
- Storage method 5

!
Note

Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Transparent Female/Male Catheter Model.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Usage.....	4
Storage method.....	5


Purpose

Training can be performed for the following items:

1. Intermittent urethral catheterization
2. Continuous urethral catheterization
3. Genital wash
4. Confirmation of indwelling balloons

Content

1. Main unit (external genitalia) 1 piece
2. Irrigator 1 bottle
3. Irrigator stand 1 piece
4. Silicon oil 1 bottle
5. Baby powder 1 bottle
6. Catheter for urethral catheterization, two-pack
(8 EG Nelaton catheter, 16 Fr Balloon catheter)

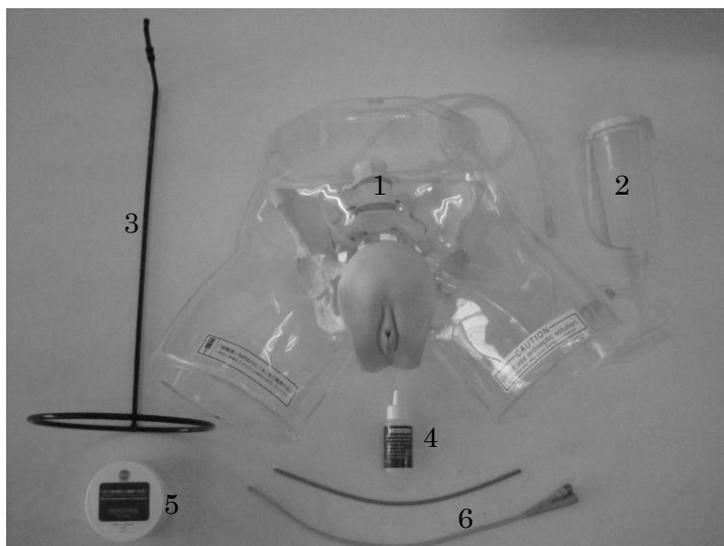

Cautions concerning handling

1. If resin products contact the external genitalia for a prolonged time, the head might be deformed.
Treat the product with sufficient care.
2. Due to the nature of the material, the external genitalia is likely to get dirty easily. Note that stains of a ballpoint pen or marker on the product cannot be removed.
3. Light dirt on the face and neck parts can be washed with neutral detergent.
(Do not use solvents, such as thinner, because they melt the resin parts.)
4. To perform training, hang the irrigator in a position approximately 30 cm higher than the urethral opening. (Use the dedicated irrigator stand.)
Hanging the irrigator in too high a position might cause a water leak.
Be careful the additional water in irrigator. If you add over the memory of 300cc of irrigator, it might cause a water leak.
5. Since the external genitalia and bladder are bonded with each other, do not remove the bladder out of the model. In addition, the external genitals are bonded with the main unit (transparent section of the body), too, so that they cannot be removed. In addition, do not remove the two tubes from the bladder. (Otherwise, a failure might occur.)
6. The transparent section of this product is easily scraped. Note that this section might be broken or fogged if hit against anything or rubbed with a strong force.
7. When performing training, wipe off water in the catheter and be sure to apply the accessory dedicated silicon oil before inserting it. Use the attached dedicated silicon oil only as a lubricant oil. (Using some other chemicals might cause a failure.)
8. If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
9. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
10. Note in advance that this product is subject to change without notice to improve it.

Usage

Bladder catheterization

1. Remove the catheter model from its case, and place it on a table with a non-slick surface.
2. Connect the tube marked red of the main unit to the irrigator.
(Refer to the right picture.)
3. Set the irrigator over the irrigator stand.
(Set it facing prominence and depression each other.)
4. Hang the tube of no mark on the side of the irrigator.(Refer to the picture.)
5. Pour the 500cc water into irrigator and open the clamp and fill up the bladder with water. Be careful the additional water in irrigator. If you add over the memory of 300cc of irrigator, it might cause a water leak.
6. Before insertion, apply the special lubricant to the end of the Nelaton catheter.
(If the catheter is inserted without applying lubricant, it will rub and damage the urethra. In addition, do not apply excessive force because if the insertion angle is incorrect, the catheter cannot be inserted.)
7. Once the Nelaton catheter reaches the inside of the bladder, the water will begin to discharge (urination) right away. Therefore, be sure to set up a tray or receptacle beforehand to catch the water discharged from the catheter.
(Refill the water inside the irrigator as necessary.)
8. Withdraw the Nelaton catheter, and begin training.

Genital wash

1. Perform training using water for the main unit.


When water is used, the surface may become sticky or not so smooth. This is no problem regarding quality; however, the surface becomes easy to tear. So, pay sufficient attention to handling.

2. Do not use any chemicals on the main unit. (Otherwise, the product might fail.)



Storage

1. When the external genitalia becomes dirty, wipe off the dirt using a soft cloth with a neutral detergent. After that, wipe it off using a cloth wetted with water.
2. Store the product after drain water in the bladder by tilting body.

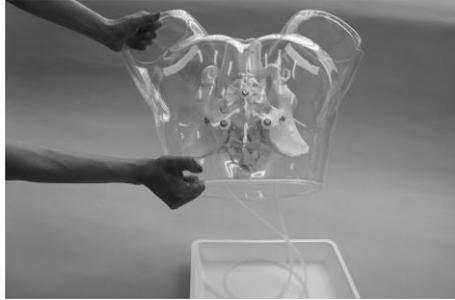

2. When storing the product in the storage case, dry the main unit adequately, and apply the silicon oil to the urethral opening and the powder to the other regions of the genitalia in advance.
3. Do not store the product in a place exposed to direct sunlight or one that becomes hot and humid. Otherwise, it might be deformed or altered.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:http://sakamoto-model.co.jp

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035,JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com