

# M165-1,2 Hemiplegia Simulation Suit M Size, L Size



SAKAMOTO MODEL  
CORPORATION

## User's Manual (151218)

### — I N D E X —

- Introduction/Table of Contents 1
- Purpose/Content 2
- Cautions on Use/Storage Instructions 3
- Instructions for Use 4

**!**  
**Note**

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.

株式会社坂本モデル



## Introduction

Thank you for purchasing our product, “Hemiplegia Simulation Suit.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

## Table of contents

|   |   |
|---|---|
| Introduction / Table of Contents.....       | 1 |
| Purpose / Content.....                      | 2 |
| Cautions on Use / Storage Instructions..... | 3 |
| Instructions for Use.....                   | 4 |

## **Purpose**

This suit is used to simulate hemiplegia (left or right).

## **Content**

- |                             |                          |
|-----------------------------|--------------------------|
| 1) Simulation suit - Qty. 1 | 5) Ankle brace - Qty. 1  |
| 2) Arm sleeve - Qty. 1      | 6) Shoes - 1 Pair        |
| 3) Wrist brace - Qty. 1     | 7) Folding cane - Qty. 1 |
| 4) Knee brace - Qty. 1      |                          |



## **Cautions on Use**

- 1) When training and simulating the experience of hemiplegia, always carry out the training with at least two assistants or supporting staff.
- 2) Be careful not to fall down when walking on roads or uneven surfaces.  
In addition, the person simulating the experience of hemiplegia may also accidentally drop objects and therefore should exercise caution.
- 3) When taking out the cane, it might extend out by itself due to the spring action. Therefore, be careful when taking it out.
- 4) The simulation suit and the arm sleeve are washable. Be sure to look at the tag on the chest area inside the suit and follow the instructions.
- 5) Note that the fasteners and printed text, etc., may become brittle after washing the suit multiple times.
- 6) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 7) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 8) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

## **Storage Instructions**

- 1) The simulation suit and the arm sleeve are washable. Be sure to look at the tag on the chest area inside the suit and follow the instructions.
- 2) Fold up the simulation suit and cane, and store them together with the other parts in the bag provided.
- 3) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.

## **Instructions for Use**

**Always carry out the training with at least two assistants or supporting staff.**

### 1) Simulation suit and arm sleeve

- The arm sleeve is attached to the left side of the simulation suit when the product is shipped. To change the arm sleeve, open the fastener and attach it to the opposite side. After putting on the wrist brace in step 2 in the “Instructions for Use”, attach the Velcro strip (hooks) on the arm sleeve to the Velcro strip (loops) on the chest area of the simulation suit.
- The Velcro strips on the arm sleeve and the chest area of the simulation suit are strong, so detach them slowly starting from the end of the strip.

### 2) Wrist brace

- Fit the wrist brace onto the wrist of the user and secure it with the belt.

### 3) Knee brace

- Fit the knee brace onto the knee of the user and secure it with the top and bottom belts.

### 4) Ankle brace and shoes

- Insert the ankle brace into the left or right shoe and secure it to the shoe.
- Put on the shoe and secure the shoe to your leg with the belt.

### 5) Folding cane

- When taking out the cane, it might extend out by itself due to the spring action. Therefore, be careful when taking it out.
- The cane length can be adjusted. Refer to the user manual provided for further details.



---

**SAKAMOTO MODEL**  
C O R P O R A T I O N

---

**Head office**

34,Higashitakagicho,Shimogamo  
Sakyo-ku, KYOTO 606-0865, JAPAN  
Phone: +81-75-701-1135  
Fax: +81-75-722-5638  
E-mail:info@sakamoto-model.com  
URL:http://sakamoto-model.co.jp

**Overseas Sales Department**

1-9-29,Senbahigashi,Minoh City,Osaka  
562-0035,JAPAN  
Phone: +81-72-729-8350  
Fax: +81-72-729-8385  
E-mail:trade@sakamoto-model.com