

M169

Fit-on Manual Disimpaction Simulator



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

● Introduction/ Table of Contents	1
● Purpose/ Content	2
● Precautions Concerning handling	3
● Storage method/ Usage	4

! Note	Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.
-------------------------	--

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Fit-on Manual disimpaction Simulator.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Storage method／Usage.....	4

Purpose

Training can be performed for the following items:

1 Manual disimpaction training

Contents

- | | |
|-----------------------------------|----------|
| 1. Main unit (cradle + anus part) | 1set |
| 2. Wearable belt | 1 unit |
| 3. Lubricant oil | 1 bottle |
| 4. Simulated excrement | 1unit |
| 5. Baby powder | 1 bottle |



Cautions concerning handling

1. The periphery of the anus on the main unit easily becomes dirty due to the material characteristics, and gets broken if rubbed excessively. Be sure to put rubber gloves on and apply the lubricant oil adequately before starting training.
2. When the model gets dirty, wipe off the dirt using a soft cloth with a neutral detergent. (Do not use solvents, such as thinner, because they melt the resin parts.) After that, wipe it off using a cloth wetted with water and apply the baby powder.
3. Use the simulated excrement and lubricant oil attached to the product.
Using something else might alter the material.
4. The periphery of the anus on the main unit easily becomes dirty due to the material characteristics, and gets broken if rubbed excessively. Be sure to put rubber gloves on and apply the lubricant oil adequately before starting training.
5. If anything strange is detected when this model is being used, stop using it and notify us as soon as possible.
6. This product is a simulator for users to perform advanced training. Nonetheless, it does not guarantee that a similar effect can be obtained by actual treatment.
7. Note in advance that this product is subject to change without notice to improve it.

Storage

1 When the anus part becomes dirty, wipe off the dirt using a soft cloth with a neutral detergent.

After that, wipe it off using a cloth wetted with water.

2 Do not store the product in a place exposed to direct sunlight or one that becomes hot and humid.

Otherwise, it might be deformed or altered.

Usage

1 Take the main unit out of the dedicated case, and set the simulated excrement.

1) Remove the cap equipped on top of the body. To do so, unlock the cap, pinch the left and right dents, and pull it out.

2) Take the simulated excrement appropriately, and insert it into the body as if pushing it in.

3) Close the cap. To do so, push in the cap deep so that it is aligned with the surface of the main unit, and lock the cap.

4) The simulated excrement's hardness varies depending on the quantity of the contained water.

Perform training in the range of suitable hardness.

2 Set the main unit.

(When the buddy wears the main unit to perform training, wind the accessory belt firmly before stating it.)

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com

URL:<http://sakamoto-model.co.jp>

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com