

M175-1 Transparent Suction Model



SAKAMOTO MODEL
CORPORATION

User's Manual (151218)

— I N D E X —

● Introduction/Table of Contents	1
● Purpose/ Content	2
● Precautions concerning handling	3
● Usage /Storage method	4

! Note	Please open your parcel and check the main unit and accessories of the product immediately upon receipt. If the products are delivered damaged or with parts missing, please contact our company or the distributor.
-------------------------	--

株式会社坂本モデル



Introduction

Thank you for purchasing our product, “Transparent Suction Model.”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction／Table of Contents.....	1
Purpose／Content.....	2
Precautions concerning handling.....	3
Usage／Storage method.....	4


Purpose

The following suctioning can be confirmed:

1. Oral and nasal suctioning
2. Suctioning through the endotracheal tube
3. Suctioning through the tracheostomy tube

Content

1. Main unit
2. Suction tube with LED light (in a dedicated case)
3. Silicone oil
4. 2 batteries (AAA alkaline batteries)

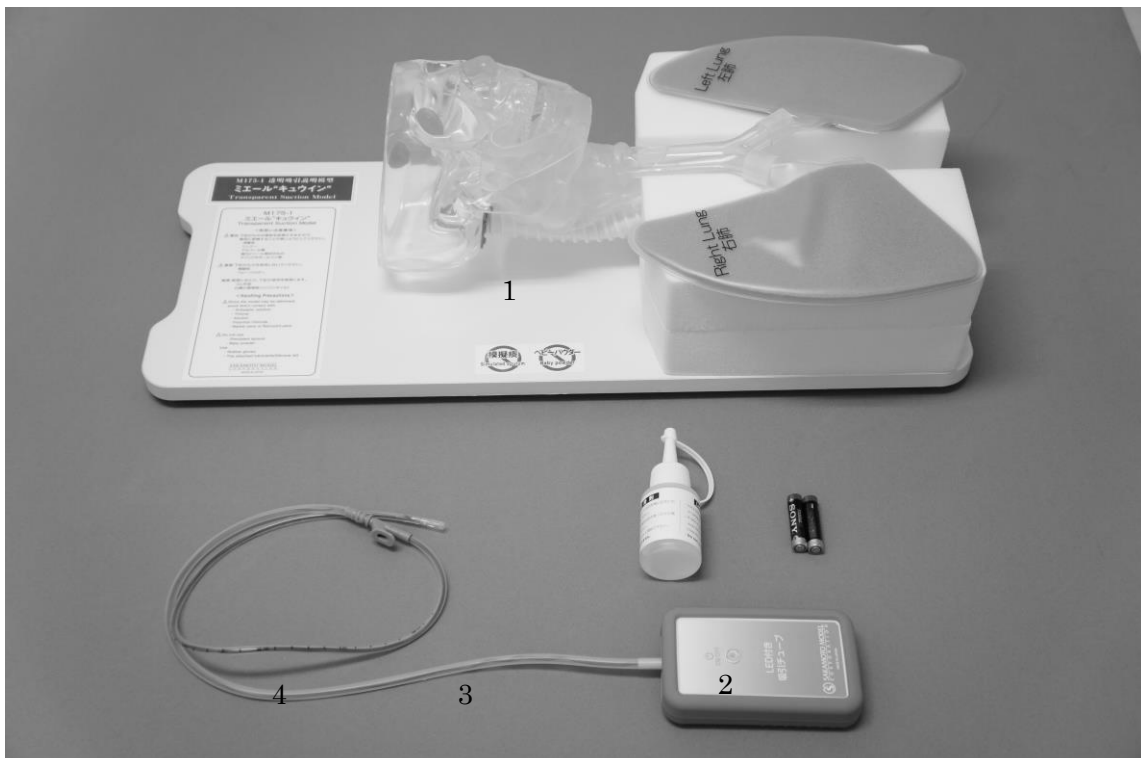

Precaution concerning handling

1. Avoid prolonged contact with resin products or solvents, as it may cause distortion of the head part of the product.
2. Due to the nature of the material, the head part is likely to get dirty easily. It is recommended to use rubber gloves (powder-free latex gloves to prevent adhesion of powders to transparent parts of the product).
Note that stains of a ballpoint pen or marker on the product cannot be removed.
3. Adhesion of powders may reduce the transparent nature of the product.
4. Avoid wetting or soaking the suction catheter with LED light, as it may cause damage.
5. Avoid unnecessary bending of the catheter part of the suction catheter with LED light, as it may cause disconnection of the catheter.
6. Dedicated silicone oil must be applied before inserting the suction catheter with LED light. Use only the dedicated silicone oil as a lubricant. (The use of other substances may cause damage to the product.)
7. Light dirt can be removed with neutral detergent. (Do not use solvents, such as thinner, because they melt the resin parts.)
8. If you notice any defect in the product, discontinue use immediately and contact our company or the distributor.
9. This product was designed to simulate advanced medical training.
However, this does not guarantee similar results in actual treatment.
10. The specifications of this product may be upgraded without prior notice

Usage

1. Take the main unit out of the hard case, and place it on a non-slip desk surface.
2. Remove the blue cover to the suction catheter (with LED), open the battery compartment in the back of the simulator and insert the batteries.
Use the two AAA size alkaline batteries. Make sure that the positive and negative sides on the batteries are inserted correctly.
Replace the batteries with new ones when the indicator light fades.
3. Apply silicone oil when using the suction catheter with LED light.

Storage method

1. Avoid storing the product in direct sunlight or in locations with a high temperature or humidity, as it may cause deformation or property change.
2. Remove the batteries when storing the product.
Leaving the batteries inside the product for a long period of time can result in corrosion, and cause a break in the electric current.
3. Avoid excessive bending of the suction catheter with LED light when storing in the dedicated case.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN
Phone: +81-75-701-1135
Fax: +81-75-722-5638
E-mail:info@sakamoto-model.com
URL:<http://sakamoto-model.co.jp>

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035,JAPAN
Phone: +81-72-729-8350
Fax: +81-72-729-8385
E-mail:trade@sakamoto-model.com