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
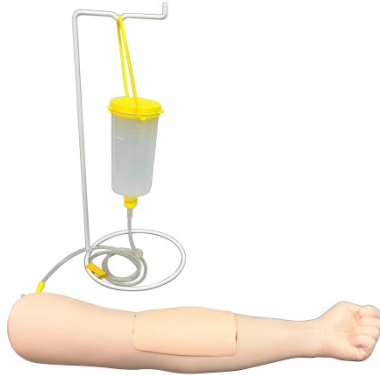
Sakamoto venipuncture trainer light

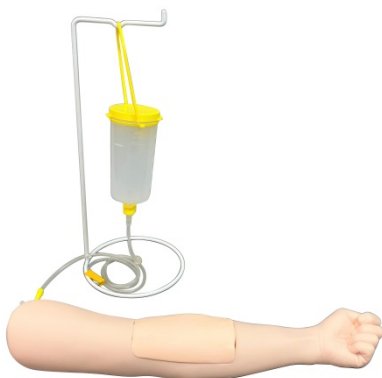User's Manual

Only use **water**^{*} or **artificial blood** for carrying out injection training using this product!

Furthermore, do not use syringes with bent needle tips!

* Use either tap water or distilled water for injection use.

Table of contents

1	Introduction	17	5	Part names	20
2	Product contents	18	6	How to use	21
3	Cautions on use	19	7	Cleaning and storage	29
4	Product applications	20	8	Specifications	30

SAKAMOTO MODEL
CORPORATION

1 Introduction

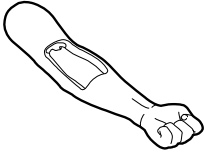
Thank you very much for purchasing the " Sakamoto venipuncture trainer light ".

This user's manual contains cautions and notes on how to use this product safely. Please read this manual carefully in order to ensure that you know how to use the product safely.


In addition, after reading the manual, keep it in a safe place where it can be referred to at any time later on.

After purchasing the product, check the package contents carefully to make sure that nothing is missing or broken. If you notice any problems, contact Sakamoto Model.


2 Product contents


Arm model ×1

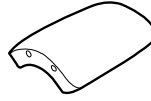
Injection pad ×1

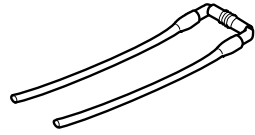
Blood vessel tubes ×1

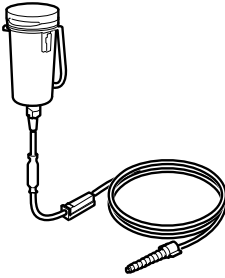
Irrigator bottle ×1

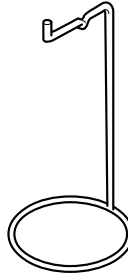Irrigator bottle
stand ×1

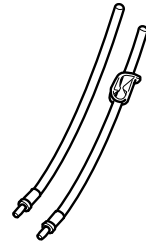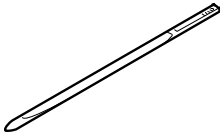
Blood vessel connection tubes
(with / without clamp) each ×1

Blood vessel threader ×1

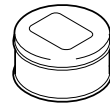Replacement blood vessels
(standard size) ×10

Baby powder ×1

3 Cautions on use

In order to use this product safely, be sure to observe the following.
(Items which are particularly important are marked with "■".)

● **Do not subject the product to impacts or other forces.**

- Do not drop.
- Do not bump against solid objects.
- Do not apply excessive loads.

If any of the above occur, damage may result.

● **Handle the injection pad gently.**

- Do not touch it with sharp objects such as knives, scissors.
- Do not pull or twist them with a strong force.

● **Always use fresh needle tips in the syringes when practising.**

If you use a syringe with a needle that has a bent or damaged tip, it may cause the blood vessels to become damaged or the injection pad to become degraded.

● **Only use water or artificial blood as the fluid for injecting.**

If a fluid other than those mentioned above is used, damage or disagreeable odors may result.

Use either tap water or distilled water for injection use. To create artificial blood, add 1 to 2 drops of the artificial blood concentrate to water.

● **Do not process or modify the product.**

Do not make holes in the product, or process or modify it so that other items can be installed to it, etc. Damage or unexpected problems may occur.

● **Do not write on the arm model with items such as ballpoint pens.**

If you write on the arm model using a ballpoint pen or similar object, the surface of the arm model will become smeared with ink which cannot be removed.

● **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.

● **There are limits to the effectiveness of simulator training.**

This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee that training using this product will provide the same level of expertise as actual treatment.

● **Do not use this product for other applications.**

Do not use this product for applications other than the original application for which it is designed. The product may become damaged or injury may result.

4 Product applications

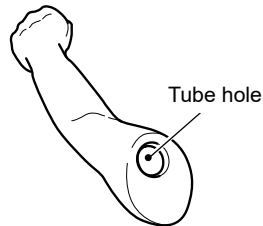
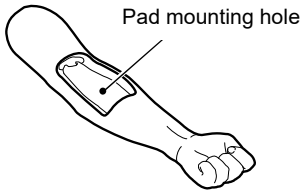
This product can be used to carry out the following training.

● **Intravenous injections**

● **Drawing blood**

5 Part names

■ **Arm model**


6 How to use

■ Preparation

Carry out the following preparations before using this product.

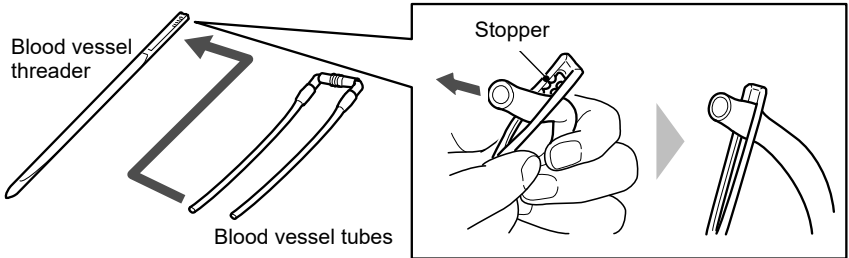
- 1 Pass the end of the blood vessel tube through the blood vessel threader.**

● TIP

Widen the hole in the blood vessel threader, and insert the end of the blood vessel tube so that it hooks onto the stopper.

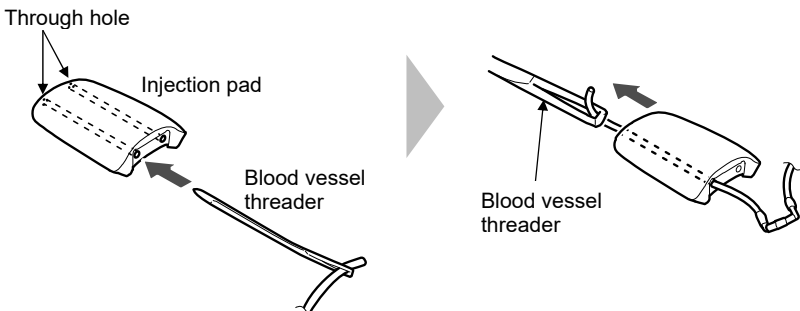
- 2 Pass the blood vessel threader into the through hole and pull the blood vessel tube through the pad.**

● TIP

If the tube is difficult to pull through, apply baby powder to the tube.

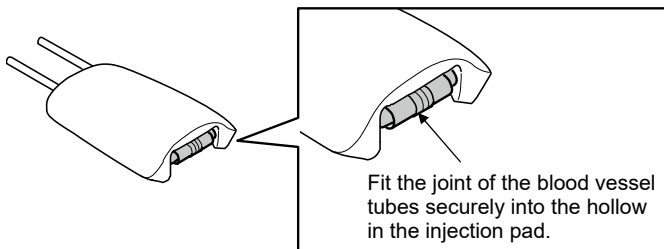
3 Repeat steps **1** to **2** for the tube on the opposite side.

TIP

If the blood vessel tubes are pulled too strongly, the injection pad may become bent. If this happens, re-insert the blood vessel tube.

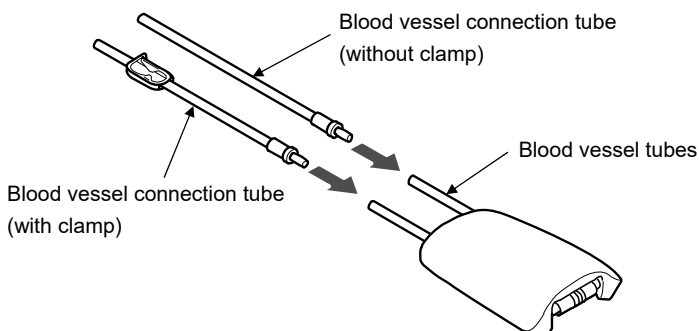
4 Insert the blood vessel connection tubes (with / without clamp) into the blood vessel tubes.


TIP

The tube with the clamp can be on either side.

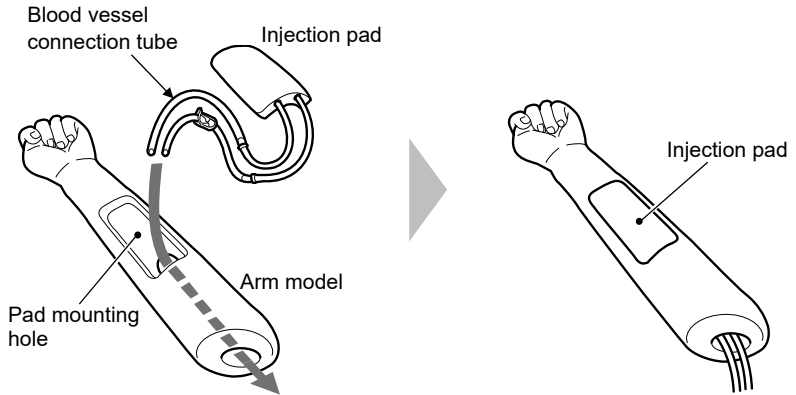
- 5** Pass the blood vessel connection tubes through the arm model, and fit the injection pad into the pad mounting hole.

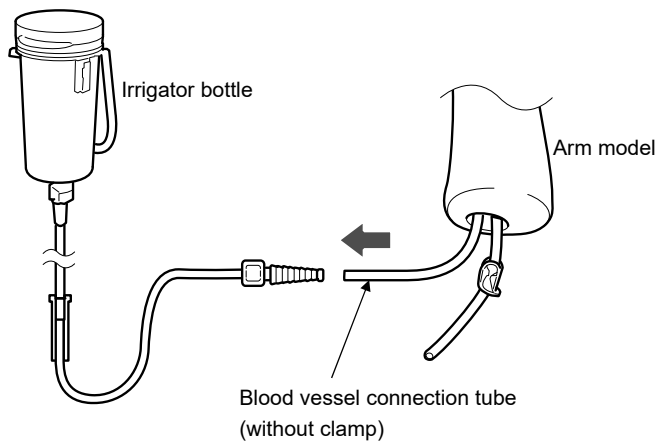
- 6** Connect the blood vessel connection tube (without clamp) to the irrigator bottle.

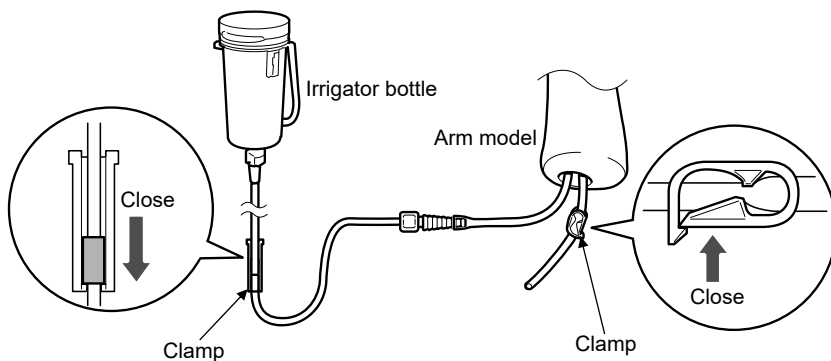- 7** Close the clamps of the irrigator bottle and blood vessel connection tube.

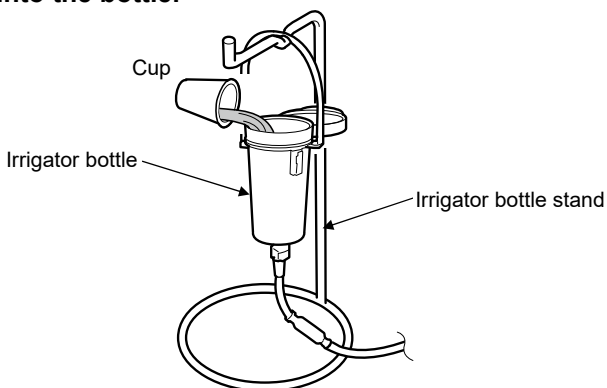- 8** Suspend the irrigator bottle on the irrigator bottle stand, and pour water into the bottle.


TIP

Suspend the irrigator bottle so that it is positioned higher than the arm model.

- 9 Open the clamps of the irrigator bottle and the blood vessel connection tube, check that the water flows out of the irrigator bottle, and then close the clamp of the blood vessel connection tube.

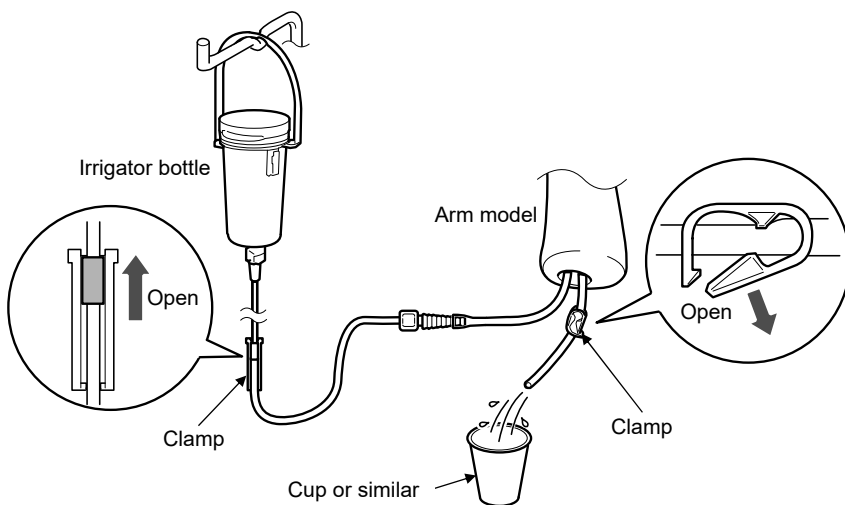Carry out training.


CAUTION


Use a cup or similar to hold the water which flows out of the blood vessel connection tube.

■ Putting away

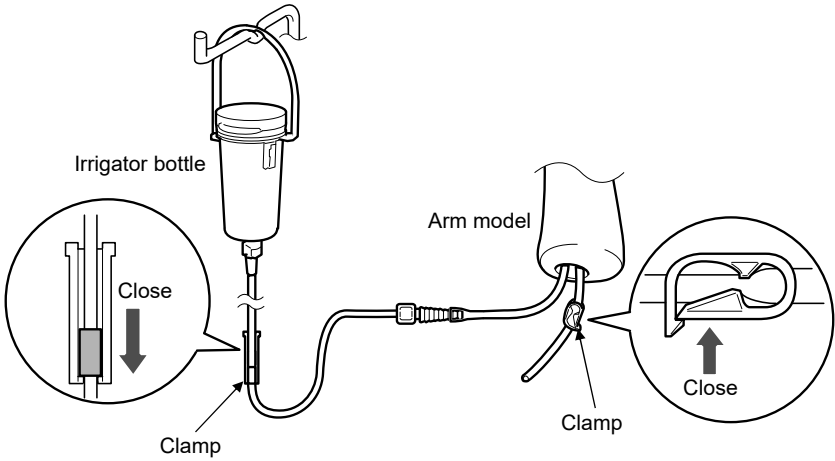
- 1 Close the clamps of the irrigator bottle and blood vessel connection tube.

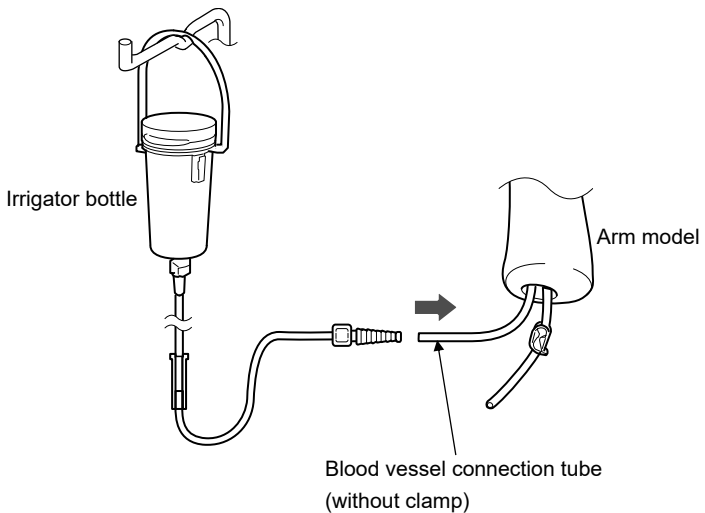- 2 Disconnect the blood vessel connection tube (without clamp) from the tube of the irrigator bottle.

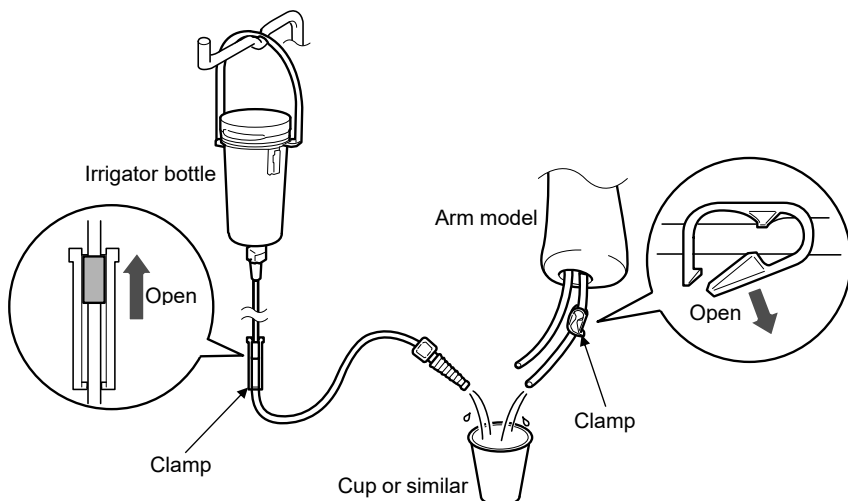- 3** Open the clamps of the irrigator bottle and the blood vessel connection tube, and release all of the water remaining in the irrigator bottle and the blood vessel connection tube.

CAUTION

Use a cup or similar to hold the water which flows out of the blood vessel connection tube.

TIP

The water inside the irrigator bottle can also be removed by opening the cap of the irrigator bottle and pouring the water out. In this case, close the clamp of the irrigator bottle.

- 4** From this point on, carry out steps **5** to **2** (in that order) in "Preparation".

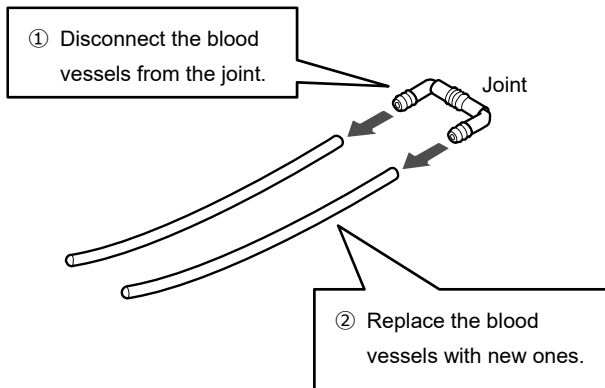
CAUTION

After training is completed, be sure to remove the blood vessel tubes from the injection pad. If the tubes are left in the injection pad, they will deteriorate.

<Replacing the blood vessel tubes>

The blood vessel tubes will deteriorate when they have been injected many times.

If this happens, replace the blood vessels with replacement blood vessels (standard size) which are provided as accessories or sold separately.

7 Cleaning and storage

Be sure to observe the following when cleaning and storing the product.

- **Clean the arm model and the surface of the injection pad body by wiping them gently with a soft cloth.**
 - If it is extremely dirty, wipe it with a cloth which has been moistened with diluted alcohol, and then wipe it again with a dry cloth.
 - Do not use benzine, thinner or other solvents. Otherwise, it may damage the product or cause the epidermis to deteriorate.
 - If you write on the product using a ballpoint pen or similar object, the surface of the product will become smeared with ink which cannot be removed.
- **Coat the injection pad with baby powder to protect it and to stop it from becoming sticky.**

If not using the product for a long period of time or if the injection pad is sticky, coat it with the accessory baby powder.
- **Do not store in places which are exposed to high temperatures, high levels of humidity or large amounts of dust.**

If the product is stored in a place which is exposed to direct sunlight, near heating appliances or in places with large quantities of oil, moisture or dust, problems with operation may result.
- **The injection pad is a consumable part.**

The injection pad will deteriorate when it has been injected many times.
In such cases, replace the injection pad with a new one (replacement part) which can be purchased separately.
- **The blood vessels are consumable items.**

If carrying out repeated injections and a large number of holes are made in the blood vessels, purchase replacement blood vessels (standard size).

8 Specifications

■ Overall product

Weight	2.5 kg
--------	--------

■ Arm model

Dimensions	W 60 cm × D 11 cm × H 12 cm (Dimensions are approximate.)
Weight	840 g

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URL <http://sakamoto-model.co.jp>

E-Mail trade@sakamoto-model.com

Product specifications and appearance are subject to change for improvement without notice.

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- Some of the components used in this product are manufactured by other companies, and product names, etc. for these components may be trademarks or registered trademarks of the manufacturers.