Z100 I. V. Pad Animal

SAKAMOTO MODEL
CORPORATION

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User's Manual (200924)

— I N D E X —

● Introduction/Table of Contents	1
● Purpose/Content	2
● Cautions on Use	3
● Replacement of blood vessels	4
● Instructions for Use	5
● Storage Instructions	6

! Note

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.

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Introduction

Thank you for purchasing our product, “I.V. PAD Animal”

Read this user’s manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

Introduction/ Table of Contents.....	1
Purpose/Content	2
Cautions on Use.....	3,4
Instructions for Use.....	5
Storage Instructions.....	6

Purpose


1 Intravenous injection training

2 Blood sampling training

Content

- 1) I.V. Pad training kit (5 pcs.) – 5 sets

1-1. I.V. Pad
1-2. Receptacle
1-3. Silicon tube with blue clasp
1-4. Silicon tube with white clasp
- 2) Exchangeable blood vessel (Narrow) – Qty. 5
Exchangeable blood vessel (Extra Narrow) – Qty. 5
- 3) Blood vessel placement stick – Qty. 1
- 4) Connector for an extra narrow vessel – Qty. 10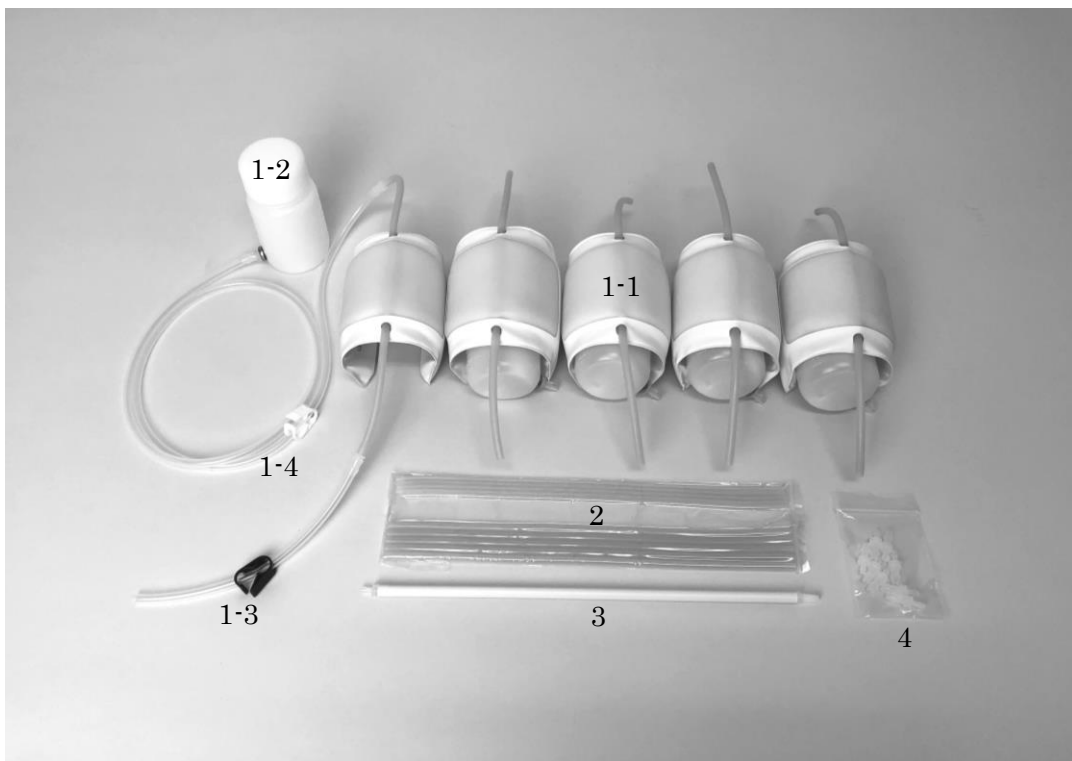

Cautions on Use

- 1) When dyeing water in order to simulate blood, use our Stock Solution for Simulated Blood (sold separately) and dilute it with water.
If a dark colored solution (higher concentration) is used, the ink may stain the injection area on the skin.
- 2) Be careful that the connector in the middle of the tube does not disconnect accidentally during use.
- 3) Use new needle tips when performing injection training with this product. Otherwise, it can cause the skin of the I.V. Pad to deteriorate.
- 4) Never inject the needle in an upright direction nor use excessive force that may pierce all the way through the I.V. Pad. Doing so is extremely dangerous and can cause serious injury.
- 5) When using this product for training purposes, conduct the training in pairs with one person playing the role of the patient and wearing the I.V. Pad.
- 6) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model.
- 7) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 8) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.
- 9) If the blood vessel is accidentally removed from the I.V. Pad:
*See the next page (Replacement of blood vessels).




Replacement of blood vessels

1 Connect the blood vessel to the placement stick and insert.
(A narrow vessel is used in this photo.)

※ When using an extra narrow vessel exchange to an extra narrow connector.
Both ends of the stick correspond to narrow
And extra narrow.

POINT

Pass between the sponge and skin sheet.

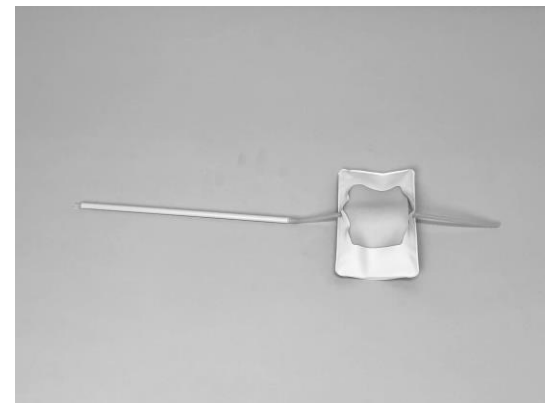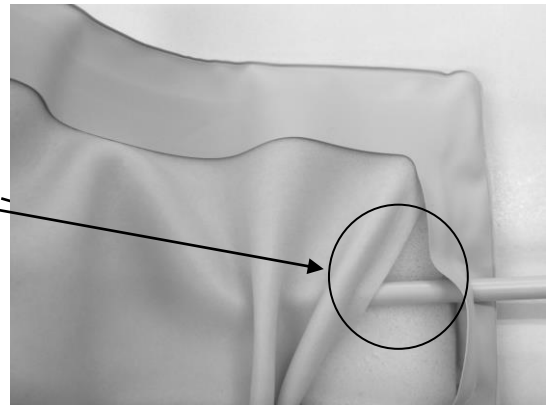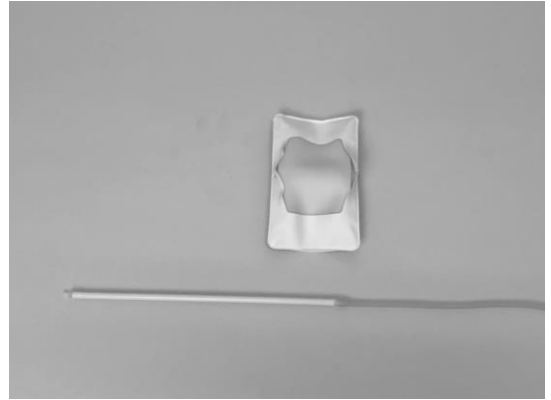
Instructions for Use

- 1) Connect the silicon tube with the white clasp to one side of the blood vessel in the I.V. Pad and to the receptacle.

- 2) Connect the silicon tube with the blue clasp to the other side of the blood vessel in the I.V. Pad.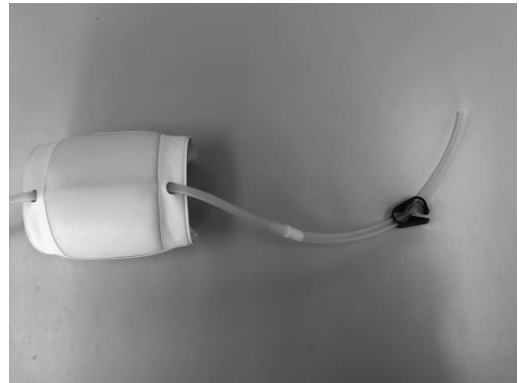

- 3) Close the blue clasp and fill up the receptacle with water (approximately 250ml).
- 4) Open the blue clasp to fill up the tube with water.
Have a tray or container ready because the water will discharge.
(Loosen the cap on the receptacle if the water does not discharge.)
- 5) The I.V. Pad uses an arm band that naturally fits and curls around your arm.
Open the arm band to fit it onto a person playing the role of the patient or onto a dummy model.
- 6) Use this product to carry out training.

※¹When exchanging the blood vessel during training, close the white clasp and disconnect the silicon tube with the white clasp from the I.V. Pad. Have a tray or container ready because the water will discharge.

Thereafter, disconnect the silicon tube with the blue clasp from the I.V. Pad, and exchange the vessel following steps 1 through 7 under the “Instructions for Use.”

※²There is a hole (1mm) in top of bottle.

Please be careful. If the bottle collapse, water is spilled.

※³If difficult to insert the connector into the blood vessel, soak water inside of blood vessel.

Storage Instructions

- 1) Close the white clasp, and disconnect the silicon tube with the white clasp from the I.V. Pad.
Have a tray or container ready because the water will discharge.
- 2) Discharge the water that is inside the tube and the receptacle.
- 3) Disconnect the silicon tube with the blue clasp.
- 4) Dry out each part at room temperature.
- 5) Roll up the silicon tube with the blue clasp and the silicon tube with the white clasp. Place them as well as the receptacle inside the I.V. Pad. Then, put the whole I.V. Pad training kit inside a bag to store it.
- 6) Avoid storing the product in direct sunlight or in locations with high temperature and humidity.

SAKAMOTO MODEL
C O R P O R A T I O N

Head office

34,Higashitakagicho,Shimogamo
Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135

Fax: +81-75-722-5638

E-mail: info@sakamoto-model.com

URL: <http://sakamoto-model.co.jp>

Overseas Sales Department

1-9-29,Senbahigashi,Minoh City,Osaka
562-0035, JAPAN

Phone: +81-72-729-8350

Fax: +81-72-729-8385

E-mail: trade@sakamoto-model.com